Ekso Walker Instruction Manual

Contents

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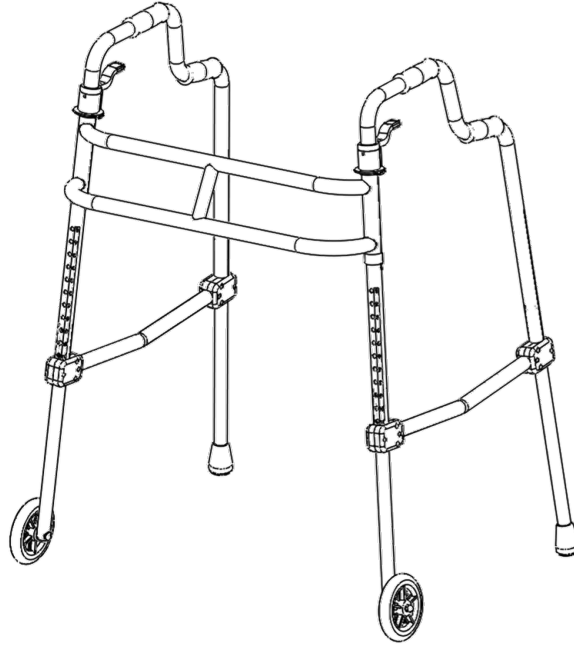
Introduction

The Ekso Walker (Model 103682) is intended for use with EksoNR and Ekso Indego products. It provides additional support for users while sitting, standing, and walking.

Specifications

Part Number 103682

The Ekso Walker has the following features:

- Aluminum frame with collapsable sides
- Sides feature two-level handles with grips for improved sitting and standing
- Front wheels
- Adjustable length legs

Weights & Dimensions

Dimensions (L x W x H) mm/in – max height	635 x 813 x 1092 / 25" x 32" x 43"
Dimensions Folded (L x W x H) mm/in	140 x 838 x 864 / 5.5" x 33" x 34"
Weight	4.1 kg / 9 lbs
Maximum user mass	136 kg / 300 lbs
Handle Height Range (min-max) mm/in	806 – 1058 / 32-42"

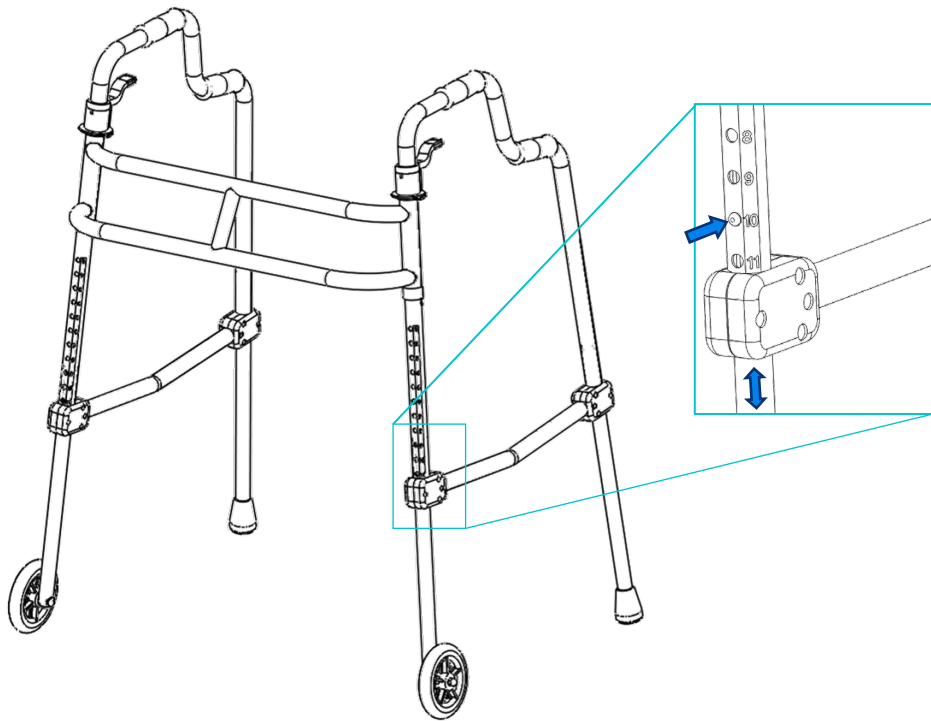
Intended Use & Intended Environment

The Ekso Walker is designed to be used by anyone needing additional stability when walking, sitting, or standing. It is specifically designed for use while using Ekso Bionics Exoskeletons.

The Ekso Walker is to be used inside.

Important Safety Information

- Ensure that the legs are adjusted to equal heights and locked in position. The adjustment button should protrude through the adjustment hole to lock the leg length.

- Ensure that the walker is fully unfolded and locked in position. The handles should be fully opened and the unlock bolt returned to flush.

Locked

Unlocked

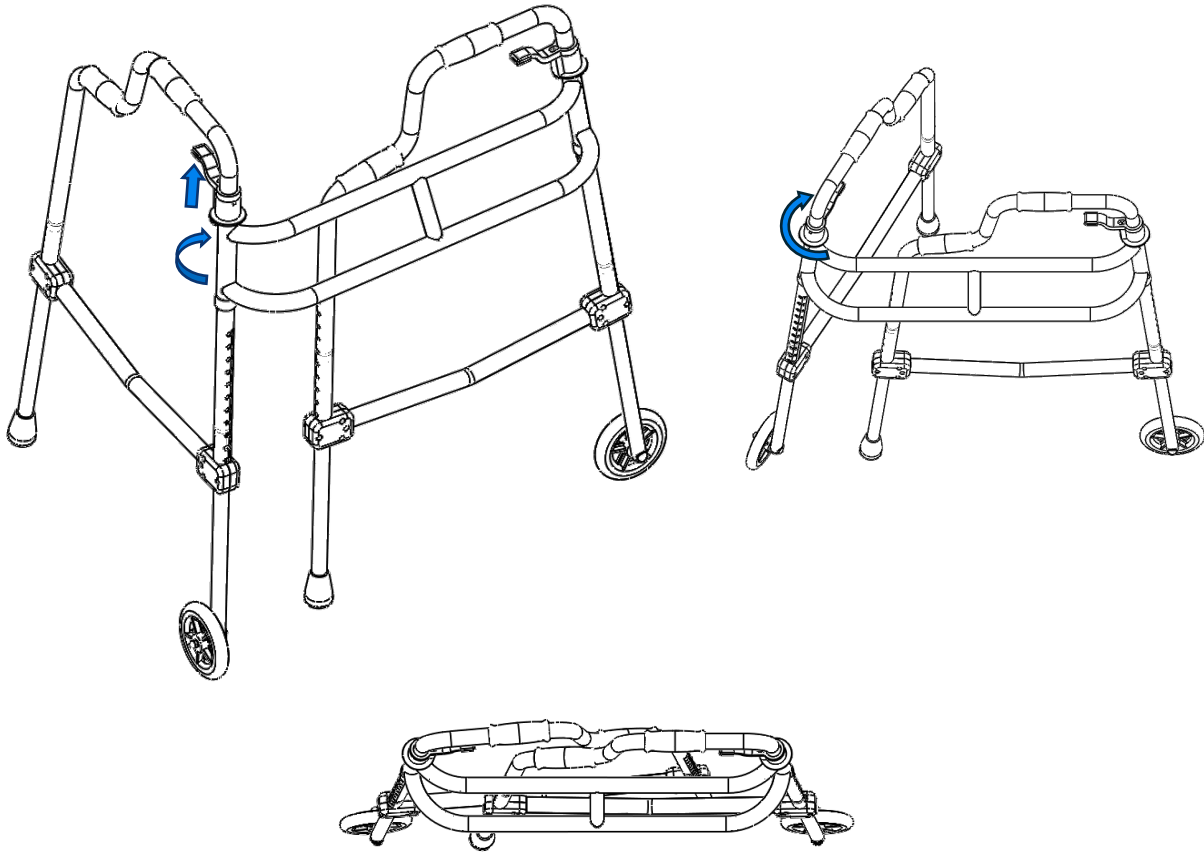
- Avoid wet or icy surfaces. Remove all obstacles from your path.
- Do not use on steep surfaces or stairs.
- Ensure that the handgrips are secure and do not twist or move.
- Ensure that the tips or wheels are not damaged or missing.
- Observe the weight limit
- Do not replace rear legs with additional wheels.
- When using in high temperatures or direct sunlight, be cautious of surface temperatures.

Setting Up the Walker

1. Remove the walker from the box
2. Unfold the walker sides and lock them into place
3. Adjust the leg lengths as appropriate for the user and ensure that they are locked in place

Storage

1. To fold the walker, lift the levers and fold in the sides. When closing, be cautious catching clothing or body parts between the two sides.

2. Walker may be carried by holding the handgrips.

Disinfecting

Contact areas of the Ekso Walker should be wiped down between users with a disposable, **non-bleach**, disinfecting towelette such as Caviwipes.

Care and Maintenance

- Wheels, locks, and adjustments should be inspected regularly and prior to first use
- Walker tips should be examined for wear. When worn, walker tips should be replaced with a rubber walker tip suitable for diameter 7/8" / 22 mm.
- Ensure that no moisture has entered the walker. If the walker has gotten wet, remove the legs and ensure that the moisture is allowed to drain fully.

If the walker is damaged, bent, or cracked, cease use and replace. Do not perform maintenance on the walker while in use.

Device Malfunctions and Serious Injuries

Incidents involving the Ekso Walker that lead or could have led to:

- life threatening or serious injury,
- death, or
- damage to property

must be reported to Ekso Bionics.

If such an event occurs, immediately cease all use of the device and report the event to Ekso Bionics. Events occurring in the European Union should also be reported to the competent authority of the Member State.

Copyright

© 2025 Ekso Bionics, Inc.

All Rights Reserved

No part of this publication may be reproduced, transmitted, transcribed, stored in a retrieval system, or translated into any language or any computer language, in any form or by any third party, without the prior written permission of Ekso Bionics, Inc.

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Ekso Bionics Europe GmbH